USER MANUAL

EN

- knee support: 929600, 941940, 941941, 941942, 941943
- elbow support: 929601
- wrist support: 941973, 929602

1. Intended use:

Please read this instruction carefully before first use and keep it for future reference.
Use the product only as instructed and for the purposes it is intended for. Do not make any modifications. The product is not a medical device.

2. Technical specifications:

Material: 100% polyester, 2 mm SBR layer

3. Rules for safe use:

Universal brace for use during daily activities and training. Provides comfortable joint support without restricting movement. Conforms to the body's anatomical structure, and the Velcro closure allows for size adjustment.
The brace should be gently stretched over the appropriate joint. It should be fitted to avoid excessive pressure. Additionally, adjust with Velcro

4. Usage:

- prevention
- strength training

5. Functions of the brace:

- helps relieve the feeling of discomfort
 - provides soothing warmth and strengthening, while not restricting movement
 - high-quality material allows the skin to breathe
 - easy to put on and take off
- The material composition ensures a perfect fit to the anatomy of the body, preventing the brace from moving.
The material ensures maximum comfort of use.

6. Storage and maintenance:

The brace should be washed by hand at a maximum temperature of 30° C using mild detergents. Do not use bleach. Rinse thoroughly. Dry at room temperature.
Do not wash or spin in a washing machine.
Detailed maintenance instructions are included on the label.

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